


LIFT AND TILT - ASSEMBLY & OPERATION INSTRUCTIONS



FURNITURE IS HEAVY, MOVING OR LIFTING SHOULD NOT BE ATTEMPTED ON YOUR OWN

PART LIST


LARGE PRODUCT BOX CONTAINS:

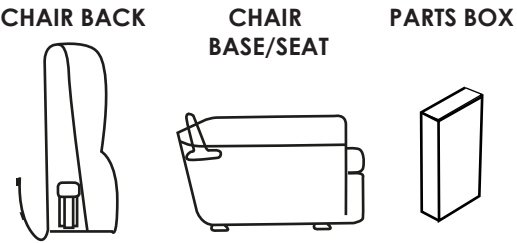


PARTS BOX CONTAINS:

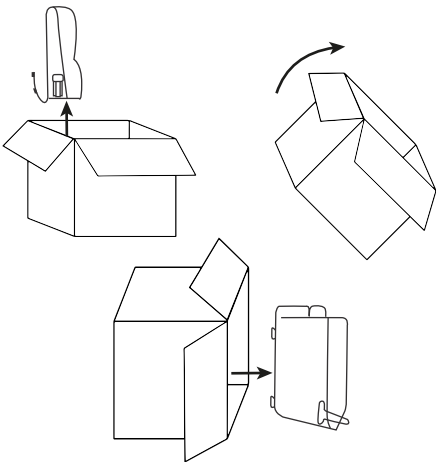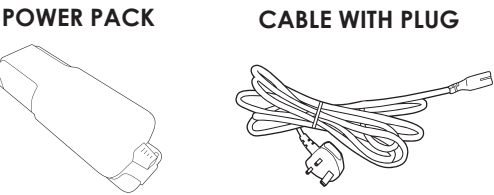



1. CAREFULLY REMOVE THE CHAIR BACK, PLACE THE BOX ON IT'S SIDE AND REMOVE THE CHAIR BASE.

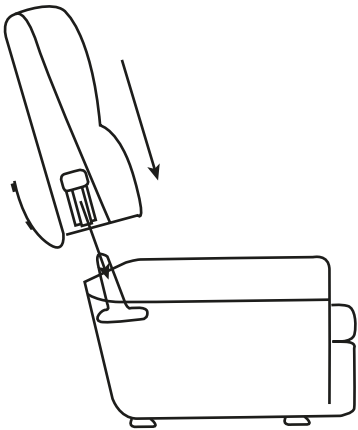


2. ALIGN PRONGS ON CHAIR BASE WITH BRACKETS ON CHAIR BACK.

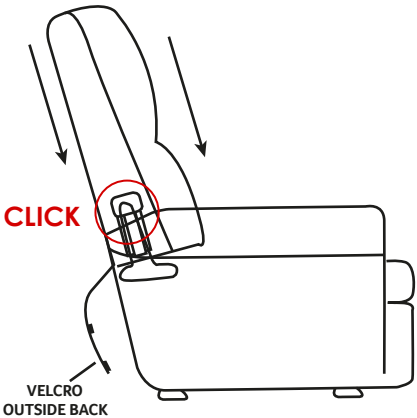


3. APPLY DOWNWARD FORCE TO THE TOP OF THE BACK ENSURING THE BRACKETS CLICK INTO PLACE.

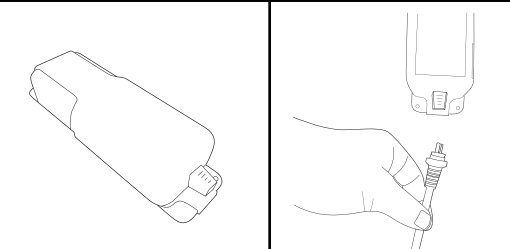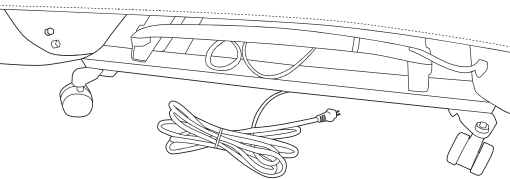


4. LOCATE THE POWER CABLE ATTACHED TO THE CHAIR BASE UNDERNEATH THE VELCRO OUTSIDE BACK. PLUG CABLE INTO POWER PACK AS SHOWN.

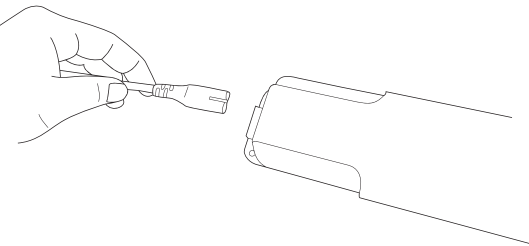

5. INSERT THE CABLE WITH PLUG INTO THE OPPOSITE END OF THE POWER PACK AS SHOWN.

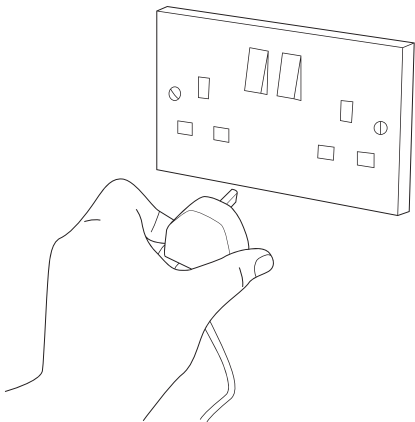

6. INSERT PLUG INTO THE CLOSEST POWER OUTLET AND SWITCH ON.

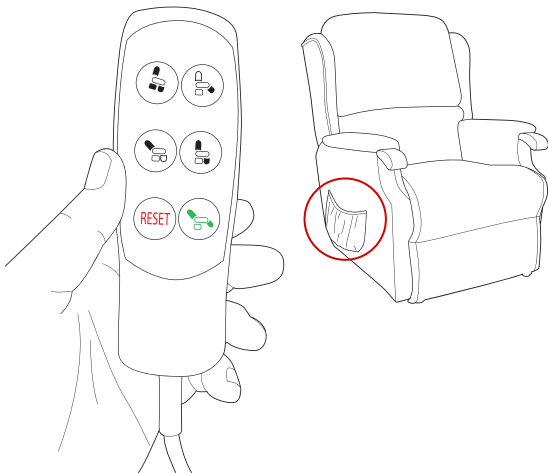

7. USE THE REMOTE LOCATED IN THE SIDE POCKET TO OPERATE CHAIR. SHADED PARTS OF BUTTONS SHOW WHICH PART OF THE CHAIR WILL MOVE. 'RESET' WILL RETURN CHAIR TO THE UPRIGHT POSITION.

PLEASE NOTE, IF YOU HAVE A WOODEN OR LAMINATE FLOOR, WE WOULD ADVISE YOU TO TAKE STEPS TO PROTECT YOUR FLOOR FROM POTENTIAL SCRATCHES AND MARKS. THE EASIEST WAY IS TO PLACE PROTECTIVE PADS UNDER THE FEET OF YOUR UPHOLSTERED FURNITURE TO PREVENT MARKS BEING MADE, OR PLACE THE UPHOLSTERED FURNITURE ON TOP OF A PROTECTIVE COVER SUCH AS A RUG. THE MANUFACTURER CANNOT BE HELD RESPONSIBLE FOR DAMAGE SUBSEQUENTLY CAUSED TO YOUR FLOORING BY NOT TAKING THE PREVENTATIVE MEASURES.